

# PROLITE® MESH•LOC™ LUMBAR SUPPORT

**Model No: 31-330**  
**Sizes: Small - 3X-Large**  
**Color: Black**

**Indications:**

- Occasional or Chronic Back Pain
- Lumbar Sprain/Strains
- Lumbar Instability
- Muscle Weakness and Postural Support

**SIZING:**

Measure around waist at the navel.

Size	Fits
Small	24 - 28"
Medium	29 - 33"
Large	34 - 39"
X-Large	40 - 44"
XX-Large	45 - 49"
XXX-Large	50 - 55"

This product is covered by one or more of the following U.S. Patents: 5,207,636 and 5,267,948.

**USE:**

This support provides abdominal compression and lift to the lumbar region, stabilizing and supporting these areas to relieve lower back pain and discomfort. Ideal for occasional or chronic back pain, lumbar sprains/strains, lumbar instability, muscle weakness and postural support.







**FEATURES & BENEFITS:**

- Patented strapping method provides hydrostatic lift to instantly relieve the load and stress on the spine
- Tapered panels contour the body for excellent compression and support
- Rigid stays provide strength and stability to the lumbar region
- Easy application with simple hook and loop closures
- Black color for extended wear

**HOW TO APPLY:**

1. Center the Mesh-Loc™ on the back so the bottom of the support is resting at the top of the buttocks. The inside size label should be at the top.
2. Wrap around and secure front closure.
3. Pull on side straps at a 45° angle to achieve lift.
4. Attach straps to the front of the Mesh-Loc™ by folding one on top of the other.

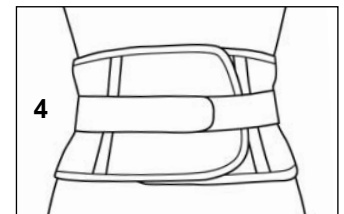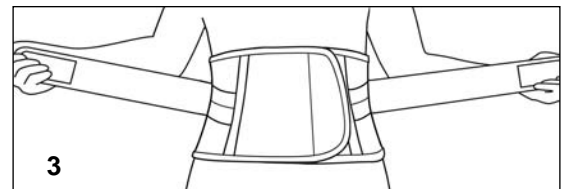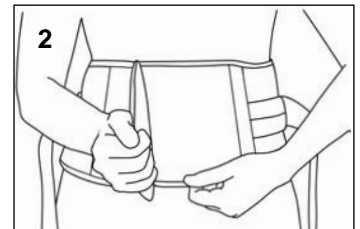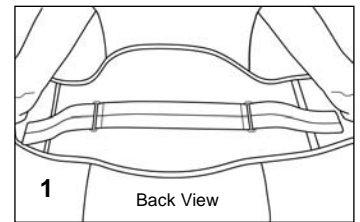

**CARE INSTRUCTIONS:**

Hand wash as needed in warm water with a mild detergent. Pat dry to remove excess water. Air dry away from heat. Do not tumble dry. Do not bleach.

Consult a physician if pain, swelling or unusual reactions are experienced. This device is only one element in the overall treatment program. There is no guarantee that injury will be cured or prevented through use of this product.



**800-327-4110**

Website: [www.flaorthopedics.com](http://www.flaorthopedics.com)

©2004 FLA ORTHOPEDICS, INC.